**Sizes:**  
 3 Women sizes:  
 WS (34 – 36.5), WM (37 – 38.5), WL (39 – 41)

4 Men sizes:  
 MS (39 – 41), MM (41.5 – 43), ML (43.5 – 45), MXL (45.5 – 47)

1 Children size (MedSurg™ Pediatric):  
 Universal (up to size 32)

**Colour:** Black

**Material:**  
 Outsole: TPR  
 Insole: EVA  
 Upper material: Polyester  
 Velcros®: Nylon

**Maintenance:**  
 The product does not require maintenance.

**Construction features:**  
 The upper material consists of breathable polyester material. The rubber sole is made of TPR with shock absorbing effect. The sole profile enables an easier rolling-off movement.

**Cleaning instructions**

**Hand washing:**

- > When the shoe is slightly dirty, clean it with a damp cloth or a soft brush.
- > The shoe may not tumbled dry or dried on the heater. Allow the shoe to air dry at moderate temperatures.

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USA / GB Instructions for use

MedSurg™ / MedSurg™ Pediatric Healing Shoe / Pediatric Healing Shoe

The MedSurg™ healing shoe provides stabilisation, shock absorption and pressure offloading after foot surgery. The shoe fits right and left equally.

1. Open the shoe by means of the velcro® straps (A, B).
2. Please ensure that the foot is securely positioned in the shoe.
3. The inner shoe flaps may overlap each other arbitrarily.
4. The optical adaption to the right or left foot can be realized by individual adjustment of the removable velcro® strap (B). The logo should be placed at the outside of the foot.
5. Close the shoe by means of the velcro® straps. If required, the length of the upper velcro® strap (A) may be shortened individually.

Indications

> Postoperative for stabilisation and offloading, e. g. foot osteotomies

**Contraindication:**  
> Open fractures

Caution

While wearing this shoe do not attempt to drive. Use extreme caution when climbing stairs (e. g. also stepping up onto curbs) and walking on uneven surfaces. Walk more slowly than usually. Recommendation to wear bandages or socks in the shoe. If the product is improperly handled its function and optimal safety can no longer be guaranteed.

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