

Body Armor® Heel Reliever Heel Protector

New
from April
2021

Effective pressure relief in the heel area for supine patients


One-Size-Fits-All



INDICATIONS

- > for the prevention and healing of heel pressure ulcers as well as diabetic foot syndrome and decubitus
- > Braden scale scores of fourteen or less
- > Diabetes with neurological sensory deficit
- > Pedal pulse not palpable
- > Paraplegia
- > prolonged immobility or coma

Available in two internal foam finishes: napped or flat

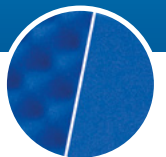

DARCO

Benefits at a glance

Forefoot strap and forefoot pad

- > keeps the foot safely in proper position
- > prevents foot drop
- > for pressure redistribution
- > forefoot pad prevents skin contact with forefoot strap

Foam with bevelled edges and corners

- > prevents pressure at the edges
- > facilitates the overlapping closure

Welded seams

- > for less irritation

Hook and loop straps

- > for easy handling
- > secures the foot in a comfortable position
- > with sewn stitched edges

Ventilation holes

- > promotes air circulation
- > helps to regulate the skin's environment

Product label

- > can be written on
- > comes with a QR code containing instructions for use

Extra tall foot section

- > accommodates larger feet
- > ensures an orthograde position
- > prevents foot drop
- > prevents foot rotation

Heel opening

- > for visual control of correct heel suspension

Long calf pad (30 cm) fixed to the inside

- > transfers the load from the heel
- > avoids hyperextension of the knee

Smooth, friction-free exterior surface

- > allows the foot to move freely in bed without affecting the suspended position of the foot



Fully adjustable
(One-Size-Fits-All)



To customise the fit, the long shin bone pad can be overlapped and fixed. Even narrow calves are kept in a stable position.

Easy customisation



In addition to the calf pad fixed to the inside, the extra foam pad for the calf and plantar foot area included with the device enables customised pressure distribution and safe application of the product.

Positioning wedge



The positioning wedge also included with the device can be used to counteract hip rotation.

Customisable for appropriate therapy

The Body Armor® Heel Reliever can easily be customised to suit the needs of each patient for targeted pressure relief.



The Body Armor® Heel Reliever redistributes heel pressure to the heel cord and gastrosoleus complex in the calf.

The heel area is completely free of all contact pressure. The heel “floats” freely in a pressure-free zone.


Article List

Body Armor® Heel Reliever

Colour: Grey-Blue

Size: Can be used universally (25 – 54 cm calf circumference, fits up to a European shoe size 47/UK shoe size 12+)


Can be worn on the right or left foot

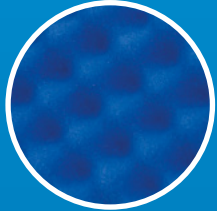

Napped foam

The convoluted foam design results in optimum air circulation and helps to ensure a pleasant cooling effect.

ORDERING CODE: NV3340 DARCO Body Armor Heel Reliever - universal, convoluted interior

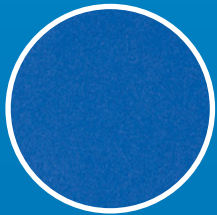

Flat foam

The smooth foam version was developed especially for sensitive skin. It is particularly suitable for Diabetes patients. This helps to prevent the formation of oedema.

ORDERING CODE: NV3340-1 DARCO Body Armor Heel Reliever - universal, smooth interior

Included in the device package



Forefoot pad



Positioning wedge



Extra foam pad



Laundry bag

The Body Armor® Heel Reliever can be washed at 60°C in the supplied laundry bag.