SlimLine®

SlimLine® Pediatric

EN Cast Boot
DE Gips- und Verbandschuh
ES Zapato para escayola y postquirúrgico
FR Chaussure pour plâtre et pansement
IT Calzatura ortopedica e coprigesso
NL Gips- en verbandschoen
DA Gips- og bandagesko
SV Gips- och förbandssko
PL But na gips i opatrunek
SL Mavčni in terapevtski čevlji
HR Gips cipela i ortopedska cipela
PT Sapato pós-operatório para gesso

DARCO

SlimLine®

SlimLine® Pediatric

EN Instructions for use

SlimLine® | Cast Boot

SlimLine® Pediatric | Pediatric Cast Boot

Important notes

If there is a serious deterioration in your state of health as a result of using the medical device, please report this to your specialist dealer or the manufacturer as soon as possible. If you report this to the specialist dealer, the dealer will forward this immediately. The reports are agreed with and processed by the competent authorities.

Please always follow these instructions for use. In case of adverse effects (e.g. skin swelling, pressure points, disorders of blood circulation, etc.) please tell your doctor. Function and optimum protection are no longer guaranteed if the product is handled incorrectly. This device assists recovery and should be used exclusively under the instruction of and in consultation with your doctor or a health-care professional. DARCO (Europe) GmbH does not assume any responsibility. Do not wear the product with injured, irritated, damaged or intact skin. You should use bandages or socks, for example.

Do not drive a vehicle while you are wearing these shoes. Use extreme caution when walking with this device on uneven surfaces. Walk more slowly than usual.

This device is intended for single-patient use only.

Purpose

The SlimLine® is a flat, versatile cast boot for the protection of all types of plaster casts and heavy compression dressings.

Indications / Contraindications

Indications

- > For plaster or cast treatments
- > For fractures
- > For postoperative care
- > Bulky dressings

Contraindications

- > Open fractures

Design features

Adult and child model

The upper is made of a breathable EVA / polyester material. The shock-absorbing sole is made of TPR rubber. The sole profile facilitates the rolling off movement.

Working life / Lifespan of the product

The lifespan of the medical device is determined by natural wear and tear due to proper handling / cleaning. The medical device is designed to be used for the usual duration of use prescribed by a doctor without showing a reduction in safety or performance. After the period of use prescribed by your doctor, check the function of the product together with your doctor or specialist dealer.

EN Use

1. Open the shoe (A).
2. Step into the open shoe.
3. Make sure that the foot is positioned securely in the shoe and the heel is as far back as possible in the shoe.
4. Close the shoe using the hook-and-loop straps (A). The foot must not be too tightly or too loosely positioned in the shoe.

Optional

The insole is removable / replaceable by specialised medical professionals.

Technical data

Sizes **5 adult sizes (SlimLine®):**
XS (33.5–37), S (37.5–39.5), M (40–41.5), L (42–44), XL (44.5–47)

3 children's sizes (SlimLine® Pediatric):
S (up to 21), M (22–26), L (27–32)

Universal fit for right and left.

Colour **Adult model (SlimLine®):** Black
Children's model (SlimLine® Pediatric): Blue or pink-blue

Material **Outsole:** TPR **Upper material:** EVA / Polyester
Insole: EVA **Hook and loop straps:** Nylon

Combination with other products

None available

Maintenance

The product does not require maintenance.

Cleaning instructions

Hand wash

- > If the shoe is slightly soiled, you can wipe it with a damp cloth or clean it with a soft brush.
- > Do not dry the shoe in a dryer or on a radiator. Leave it to dry at room temperature.

Disposal

Dispose of after use. Please dispose of the packaging in an environmentally friendly manner.
Dispose of this product after use in accordance with applicable local laws and regulations.

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DARCO International, Inc.
ALBO-Healthcare GmbH
V-M Orthotics Ltd.