Body Armor® Embrace

EN Ankle joint orthosis
DE Sprunggelenksorthese
ES Ortesis para articulación de tobillo
FR Orthèse de cheville
IT Ortesi per l'articolazione della caviglia
NL Enkelorthese
DA Ankelledsortese
SV Fotledsortes
PL Orteza stawu skokowego
SL Ortoza za gleženjski sklep
HR Ortoza za zglobov gležnja
PT Ortótese do tornozelo

DARCO

 **ALGEOS**

A. Algeos Ltd
Sheridan House
Bridge Industrial Estate
Spote Hall Road
Liverpool
L24 9HB, UK

+44 (0)151 448 1220
+44 (0)151 448 1008
algeos.com
info@algeos.com

1

4

2

5

3

5a

3a

6

A

B

EN Instructions for use

Body Armor® Embrace | Ankle joint orthosis

Important notes

Should any serious incident occur in relation to this device it should be reported to the manufacturer at the contact information listed on this document and the proper authority of the country or state where you are located.

Please always follow these instructions for use. Do not over tighten – this could lead to decreased circulation. If you experience pain, abnormal swelling or skin irritation, discontinue use and contact your medical provider. If the product is handled incorrectly, its function and optimum protection can no longer be guaranteed. This aid assists recovery and should be used exclusively under the instruction of and in consultation with your doctor or a healthcare professional.

V-M Orthotics Ltd does not assume any responsibility. Always wear socks with this orthosis. Walk more slowly than usual. This aid is intended for single-patient use only.

Purpose

The Body Armor® Embrace is an ankle joint orthosis to stabilise the ankle joint during the acute phase of injury as well as during the process of rehabilitation.

Indications / Contraindications

Indications

- > Distortion trauma of the upper ankle joint
- > Insufficiency of the capsules and ligaments of the upper ankle joint
- > Prevention of chronic instability

Contraindications

- > Sensorimotor dysfunctions
- > Open fractures
- > Ulcera cruris

Design features

The anatomically moulded external shell adjusts perfectly to the shape of the foot. The closing straps provide comfort and stability. The Body Armor® Embrace ankle joint orthosis can be individually customised to suit the foot / ankle.

Technical data

Sizes	S (2.5–6 UK shoe size) M (6.5–10 UK shoe size) L (10.5–13.5 UK shoe size)	The ankle joint orthosis is available for the right and left foot equally.
Colour	Black	
Material	Orthosis: Polypropylene, Fibreglass, EVA, Steel Softpad: Polyethylene, Polyester Bungee closure system: Nylon	

Combination with other products

No specifications

Maintenance

The product does not require maintenance.

1. Check whether your shoe has a removable insole.
 - a. If your shoe has a removable insole or footbed (as shown in image 1). Place the Body Armor® Embrace underneath the insole or footbed. In this case we recommend removing the enclosed black foam pad from the heel and not using it.
 - b. If the insole of your shoe cannot be removed, take off the adhesive backing of the enclosed black foam pad from the heel and fix the pad centrally to the upper surface of the bottom shell (as shown in image B).
2. Open the strapping system in order to simplify entering the orthosis (as shown in image 2).
3. Insert your Body Armor® Embrace into the shoe for the stirrup to assume the following position as shown in image 3 and 3a (broad sole cupping underneath the heel and opening at the outside of the foot). Please ensure that the ankle joint orthosis fits securely around your ankle and foot. Try to place the orthosis sole as far to the back of the shoe as possible.
4. Close the long lower hook and loop strap first (runs circular through the loop at the inside) by fixing it as low as possible at the front of the splint (as shown in image 4).
5. Next fasten the upper strap by laying it across the touch and close spots on the inside of the device (as shown in image 5). If the strap is too long you can shorten it with

- scissors (see image 5a). Unfasten the double-catching end of the bungee closure system (y-form / with logo) and cut off the hook and loop strap to the desired length. Afterwards fix the hook strap to the loop strap again.
6. Then tie your shoes as usual (as shown in image 6).

Use of the orthosis for preventive care after the acute phase of injury / customisation of the Body Armor® Embrace:

If you wear the ankle joint orthosis for prevention or if you are able to resume your regular daily routine after an acute injury, you can remodel your Body Armor® Embrace as follows in order to achieve a more comfortable fit:

- > Remove the foam cushions from the lateral fixing points
- > Shorten their length by cutting off the thin lower end at the pre-shaped edge (see image A).
- > Reinsert the cushion after shortening and apply the orthosis as usual.

Note

For maximum stability we recommend wearing athletic shoes / trainers. For advanced comfort please wear a high cut sock with a high absorptive capacity.

Working life / Lifespan of the product

The lifespan of the medical device is determined by natural wear and tear due to proper handling / cleaning. The medical device is designed to be used for the duration of use prescribed by a doctor without showing a reduction in safety or performance. After the period of use prescribed by your doctor, check the function of the product together with your doctor or medical provider.

Cleaning instructions

For cleaning we recommend washing the foam pads by hand with warm water and mild soap or detergent. The plastic parts of the orthosis can be cleaned with a damp cloth. Avoid damaging the material by closing all touch and close straps before washing the orthosis. The orthosis may not be tumbled dry or dried on a heater. Please allow the orthosis to air dry at moderate / room temperature.

Disposal

Dispose of after use. Please dispose of the packaging in an environmentally friendly manner. When treatment is complete, dispose of this device according to local laws and ordinances.

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