AirTraveler Walker

EN Lower Leg Foot Orthosis
DE Unterschenkel-Fußorthese
ES Ortesis de pierna y pie
FR Orthèse de jambe-pied
IT Ortesi per piede-gamba
inferiore
NL Onderbeen-voetorthese
DA Underbens-fodortose
SV Underben-fotortos
PL Orteza podudzia i stopy
SL Ortoza meča-stopalo
HR Ortoza za goljenicu i stopalo
PT Órtese de pé e perna inferior

DARCO

AirTraveler Walker

EN Date of manufacture
 DE Herstellungsdatum
 ES Fecha de fabricación
 FR Date de fabrication
 IT Data di fabbricazione
 NL Productiedatum
 DA Fremstillingsdato
 SV Tillverkningsdatum
 PL Data produkcji
 SL Datum izdelave
 HR Datum proizvodnje
 PT Data de fabrico

EC REP

EN EU Authorised Representative
 DE Bevollmächtigter in der EU
 ES Representante autorizado en la UE
 FR Représentant autorisé UE
 IT Rappresentante UE autorizzato
 NL Wettelijke vertegenwoordiger EU
 DA EU-autoriseret repræsentant
 SV Behörig representant i EU
 PL Autoryzowany przedstawiciel w UE
 SL Pooblaščen zastopnik EU
 HR Ovlašteni zastupnik u EU
 PT Mandatário na UE

UK REP

EN UK Responsible Person
 DE Verantwortliche Person im Vereinigten Königreich
 ES Persona responsable en GB
 FR Personne responsable Royaume-Uni
 IT Persona responsabile nel Regno Unito
 NL Verantwoordelijke rechtspersoon VK
 DA Ansvarlig i Storbritannien
 SV Ansvarig person för Storbritannien
 PL Osoba odpowiedzialna w Wielkiej Brytanii
 SL Odgovorna oseba v Združenem kraljestvu
 HR Odgovorna osoba u UK
 PT Responsável no Reino Unido

EN Observe the instructions for use
 DE Gebrauchsanweisung beachten
 ES Observar las instrucciones de uso
 FR Tenir compte du mode d'emploi
 IT Consultare le istruzioni per l'uso
 NL Raadpleeg de gebruiksaanwijzing
 DA Følg brugsanvisningen
 SV Läs bruksanvisningen
 PL Przestrzegać instrukcji użycia
 SL Upoštečajte navodila za uporabo
 HR Pridržavati se uputa za uporabu
 PT Consultar as instruções de utilização

EN Certification mark declaring this product's compliance with EU legislation
 DE Zertifizierungszeichen, das die Übereinstimmung dieses Produktes mit der EU-Gesetzgebung bestätigt
 ES Marca de certificación que declara la conformidad de este producto con la legislación de la UE
 FR Marque de certification attestant de la conformité de ce produit à la législation de l'UE
 IT Marchio di certificazione che dichiara la conformità del prodotto con la legislazione UE
 NL Certificeringskenmerk dat aangeeft dat dit product voldoet aan de wetgeving van de EU
 DA Certificeringsmærke, der erklærer dette produkts overensstemmelse med EU-lovgivningen
 SV Märket deklarerar att produkten uppfyller lagarna i EU
 PL Znak certyfikacyjny deklarujący zgodność tego produktu z przepisami UE
 SL Certifikacijska oznaka, ki potrjuje skladnost tega izdelka z zakonodajo UE
 HR Certifikacijska oznaka koja označava skladnost ovog proizvoda sa zakonodavstvom EU
 PT Marca de certificação que declara a conformidade deste produto com a legislação da UE

EN Certification mark declaring this product's compliance with UK legislation
 DE Zertifizierungszeichen, das die Übereinstimmung dieses Produktes mit der Gesetzgebung des Vereinigten Königreiches bestätigt
 ES Marca de certificación que declara la conformidad de este producto con la legislación de GB
 FR Marque de certification attestant de la conformité de ce produit à la législation de l'RU
 IT Marchio di certificazione che dichiara la conformità del prodotto con la legislazione del Regno Unito
 NL Certificeringskenmerk dat aangeeft dat dit product voldoet aan de wetgeving van de VK
 DA Certificeringsmærke, der erklærer dette produkts overensstemmelse med UK-lovgivningen
 SV Märket deklarerar att produkten uppfyller lagarna i Storbritannien
 PL Znak certyfikacyjny deklarujący zgodność tego produktu z przepisami Wielkiej Brytanii
 SL Certifikacijska oznaka, ki potrjuje skladnost tega izdelka z zakonodajo Združenega kraljestva
 HR Certifikacijska oznaka koja označava skladnost ovog proizvoda sa zakonodavstvom UK
 PT Marca de certificação que declara a conformidade deste produto com a legislação do Reino Unido

EN Instructions for use

AirTraveler Walker | Lower Leg Foot Orthosis

Important notes

If there is a serious deterioration in your state of health as a result of using the medical device, please report this to your specialist dealer or the manufacturer as soon as possible. If you report this to the specialist dealer, the dealer will forward this immediately. The reports are agreed with and processed by the competent authorities.

Please always follow these instructions for use. In case of adverse effects (e.g. skin swelling, pressure points, disorders of blood circulation, etc.) please tell your doctor. Function and optimum protection are no longer guaranteed if the product is handled incorrectly. This device assists recovery and should be used exclusively under the instruction of and in consultation with your doctor or a healthcare professional. DARCO International Inc. does not assume any responsibility. Do not wear the product in direct contact with injured, irritated, damaged or broken skin. This device is not recommended for direct skin contact. You should use bandages, dressings or socks.

Do not drive a vehicle while you are wearing this device. Use extreme caution when climbing stairs (e.g. also stepping up onto curbs) and walking on uneven surfaces. Walk more slowly than usual. Be careful not to inflate the air chambers too forcefully. Excessive pressure on the lower leg should be avoided!

Caution is required in patients with neuropathy to ensure that the walker does not restrict blood circulation. Patients should call their physician immediately if any tingling, burning, numbness or foul odour becomes noticeable. This device is intended for single-patient use only.

Purpose

Lower leg foot orthosis for immobilization in a given position

Indications/Contraindications

Indications

- Lower leg-foot region for:
- > Postoperative use
 - > Trauma care
 - > Stress fractures
 - > Soft tissue injury

Contraindications

- > Open fractures
- > Poor circulation

Design features

Rocker outsole with lateral stabilization elements for immobilization. Closing system with variable hook and loop fasteners. Inflatable liner in both high and low versions for individual adjustment. The lower leg foot orthosis is available in five sizes with a universal fit for the right and left.

Working life / Lifespan of the product

The lifespan of the medical device is determined by natural wear and tear due to proper handling / cleaning. The medical device is designed to be used for the usual duration of use prescribed by a doctor without showing a reduction in safety or performance. After the period of use prescribed by your doctor, check the function of the product together with your doctor or specialist dealer.

1. Open the hook and loop straps, remove the inner shoe of the walker and take out the hexagonal foam pad.
2. Place the inner shoe around the leg and fasten it with the hook and loop straps.
3. To enter the walker safely, spread the medial and lateral splints using both hands.
4. After entry, remove the white protective strips on the inside of the splint and fix the splint to the inner shoe. Now fasten the hook and loop straps, starting with the straps close to the ankle. Ensure that the leg is comfortable in the walker.
5. Now pump air through the round pump on the front of the walker into the air chambers of the inner shoe. The air is distributed evenly. The degree of compression obtained should ensure that neither the foot nor the ankle can move freely. The inner shoe of the walker should secure the foot and ankle adequately. If impaired sensation, swelling or pain occurs, the pressure should be reduced by releasing air from the chambers using the silver valve. Should persistent pain occur, the treatment should be interrupted and a doctor should be consulted.

Caution

Do not overinflate.

Optional: Insertion of the foam pad (supplied as an accessory)

The four hexagonal pads are used to immobilise and pad the ankle, thus allowing individual fitting. Place pads in areas where friction needs to be reduced or excessive space needs to be filled.

Technical data

Sizes

- > 5 Sizes: XS (up to 36), S (37 – 38), M (39 – 41), L (42 – 44), XL (45 – 47)
- > Available in knee-high and mid-calf versions
- > Universal fit for right and left

Material

Outer sole: TPR
Inner sole: EVA
Inner material: Polyester
Shaft: Nylon

Colour

Black

Combination with other products

No specifications

Maintenance

The product does not require maintenance.

Cleaning instructions

- > When the shell or inner liner is slightly dirty, clean it with a damp cloth or a soft brush.
- > Allow the shell or inner liner to air dry only.

Disposal

Please dispose of the packaging in an environmentally friendly manner. Dispose of this product after use in accordance with applicable local laws and regulations.

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A. Algeo Ltd
DARCO (Europe) GmbH
DARCO International, Inc.