# CBO + Cast Boot Pediatric

EN Cast Boot  
DE Gipsschuh  
ES Zapato para escayola  
FR Chaussure pour plâtre  
IT Calzatura coprighesso  
NL Gipsschoen  
DA Gipssko  
SV Gipssko  
PL But na opatrunek gipsowy  
SL Mavčni čevlji  
HR Gips cipela  
PT Sapato para gesso

**DARCO**

## CBO

## Cast Boot Pediatric

## EN Instructions for use

**CBO** | Cast Boot

**Cast Boot Pediatric** | Pediatric Cast Boot

### Important notes

Should any serious incident occur in relation to this device it should be reported to the manufacturer at the contact information listed on this document and the proper authority of the country or state where you are located. When treatment is complete, dispose of this device according to local laws and ordinances.

Please always follow these instructions for use. In case of adverse effects (e.g. skin swelling, pressure points, disorders of blood circulation etc.) please tell your doctor. Function and optimum protection are no longer guaranteed if the product is handled incorrectly. This aid assists recovery and should be used exclusively under the instruction of and in consultation with your doctor or a healthcare professional. DARCO (Europe) GmbH does not assume any responsibility. Do not wear the product with injured, irritated, damaged or broken skin. You should use bandages or socks, for example.

We recommend that you do not drive a vehicle while you are wearing this aid. Use extreme caution when climbing stairs (e.g. also stepping up onto curbs) and walking on uneven surfaces. Walk more slowly than usual.

This aid is intended for single-patient use only.

## Purpose

- > Cast shoe
- > To protect bulky dressings

## Indications / Contraindications

### Indications

- > Casts for fractures

### Contraindications

- > Open fractures

## Design features

Upper material with adjustable hook and loop straps. Wide opening for placement of bulky casts. The sole profile facilitates the rolling off movement.

## Combination with other products

No specifications

## Maintenance

The product does not require maintenance.

## EN Use

1. Open the shoe (A).
2. Step into the open shoe.
3. Make sure that the foot is positioned securely in the shoe and the heel is as far back as possible in the shoe.
4. Close the shoe using the hook and loop straps (A) on the heel and over the instep. The foot must not be too tightly or too loosely positioned in the shoe.
5. The hook and loop strap can be shortened / cut for a customized fit.

## Technical data

### Sizes

#### Five adult sizes (CBO):

XS (33.5–37.0), S (37.5–39.5), M (40.0–41.5), L (42.0–44.0), XL (44.5–47.0)

#### Two child sizes (Cast Boot Pediatric):

XS (to 20), S (21–22)

Universal fit for right and left

### Colour

#### Adult model (CBO):

Blue

**Pediatric model (Cast Boot Pediatric):** Multi-coloured (fabric design differences possible depending on production)

### Material

#### Outsole (CBO):

EVA

#### Outsole (Cast Boot Pediatric):

TPR

#### Inner lining:

Cotton

#### Hook and loop straps:

Polyester

#### Upper material:

Cotton, nylon

## Working life / Lifespan of the product

The lifespan of the medical device is determined by natural wear and tear due to proper handling / cleaning. The medical device is designed to be used for the duration of use prescribed by a doctor without showing a reduction in safety or performance. After the period of use prescribed by your doctor, check the function of the product together with your doctor or medical provider.

## Cleaning instructions

### Hand wash

- > If the shoe is slightly soiled, you can wipe it with a damp cloth or clean it with a soft brush.
- > Do not dry the shoe in a dryer or on a radiator. Leave it to dry at room temperature.

## Disposal

Dispose of after use. Please dispose of the packaging in an environmentally friendly manner. The product can be disposed of with household waste.

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DARCO International, Inc.
A. Algeos Ltd